

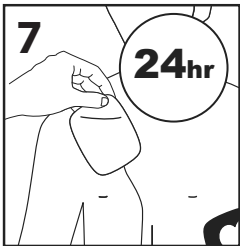
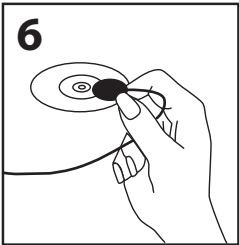
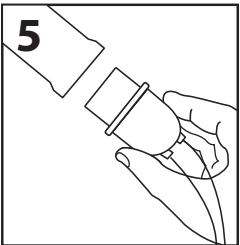
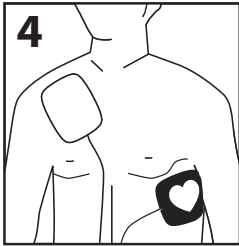
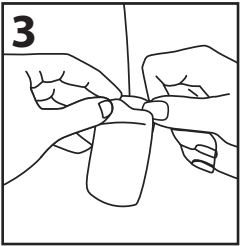
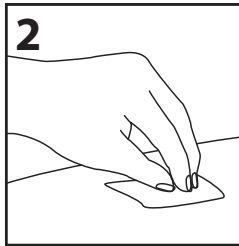
**Multi-function Defibrillation Pads**  
Monophasic or Biphasic Compatible  
Defibrillation • Cardioversion • Pacing • ECG Monitoring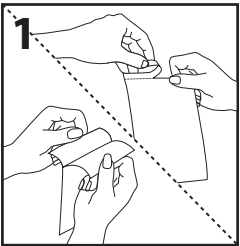

**Graphic Controls**  
d.b.a. Vermed  
400 Exchange Street  
Buffalo, NY 14204  
Vermed.com

Rev Date: 11/7/2018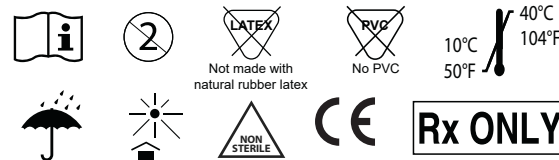

**EN ENGLISH**

**Adult/Child Defibrillator Electrodes and Pediatric Defibrillator Electrodes**

**Instructions For Use**

1. Remove Electrodes from pouch.
2. Ensure skin site is clean and dry.
3. Remove Electrodes from release liner.
4. Apply electrodes firmly to patient's skin.
5. Connect electrodes to defibrillator.
6. Connect ECG leads for demand pacing.
7. Apply a new set of electrodes after:  
24 hours on skin or 50 defibrillation shocks.

**Warnings**

- Do not use if conductive polymer gel is dried or damaged.
- Misuse of electrodes may cause patient burns.
- Do not use if electrodes are damaged.
- Not compatible with all models. Check compatibility before use.

**Cautions**

- Do not open package until immediately prior to using electrodes.
- Electrodes may dry out when removed from packaging and exposed to air.
- If electrodes do not adhere properly to patient, apply a new pair.
- Do not apply any substance to the skin surface that will leave residue.
- Do not apply electrodes over broken or damaged skin.
- Do not crush, fold or store package under heavy objects.
- Pacing requires separate leads and electrodes for monitoring.
- The adhesive foam may be a mild irritant to the skin.
- Electrode placement diagrams are suggestion only. Follow hospital protocol.
- Pacing, defibrillation or cardioversion may cause reddening of the skin.

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